


OPERATING AND MAINTENANCE MANUAL

HOSE AND HOSE ASSEMBLY

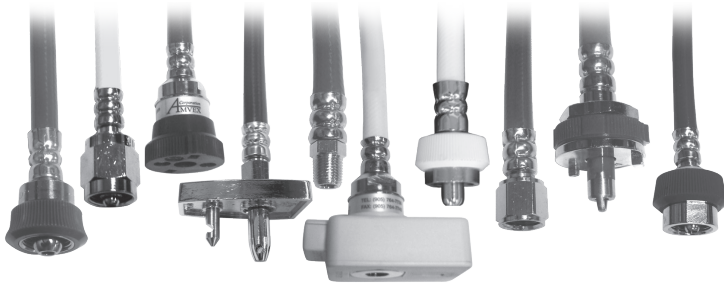

Model Number:
HS-XXYY-XXYYXX

Length in feet

Color code

Gas

C4:Conductive 1/4"

C5:Conductive 5/16"

N4:Non-Conductive 1/4"

N5:Non-Conductive 5/16"

End Connection

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.



25B Ea st Pearce St., Richmond Hill, ON L4B 2M9 Canada | Tel: 905-764-7736 | Fax: 905.763-7743 | www.amvex.com

1111 Lakeside Drive, Gurnee, IL 60031-4099
Tel: 1-800-448-0770 | Fax: 847-855-6300
www.ohiomedical.com

IMPORTANT: SAFETY INSTRUCTIONS

This manual is to provide you with information on the Hose Assembly and to ensure the proper use and safety for this product.

READ AND UNDERSTAND ALL THE SAFETY AND OPERATING INSTRUCTIONS CONTAINED IN THIS BOOKLET.

IF YOU DO NOT UNDERSTAND THESE INSTRUCTIONS, OR HAVE ANY QUESTIONS, CONTACT YOUR SUPERVISOR, DEALER OR THE MANUFACTURER BEFORE ATTEMPTING TO USE THE APPARATUS.

⚠ WARNING: Indicates a potentially hazardous situation, which, if not avoided, could result in death or serious injury.

ATTENTION: Indicates a potentially hazardous situation, which, if not avoided, could result in minor or moderate injury.

CAUTION: Indicates a potentially hazardous situation, which, if not avoided could result in property damage.

Receiving Inspection

Remove product from package and inspect for damage. Verify that the model received is in working order. If product is damaged, or incorrect do not use. Contact your dealer, equipment provider or manufacturer.

User Responsibility

This product performs as explained in this manual. This holds true as long as the assembly, use, repair and maintenance are properly followed according to our instructions. Periodic review of this device is recommended. If any damage or defects are present, the product should not be used. This includes parts that may have been altered, contaminated, worn or missing. If any of the above are noted, immediate repair/replacement is required. Amvex Corporation recommends the hose assembly undergo visual inspection every 5 years. It is also recommended that the Hose Assembly be replaced within 10 years from date of which it was manufactured.

In compliance with the Amvex Warranty, repair of this device is not to be performed by anyone other than an Amvex trained professional and done in strict accordance to the written instructions provided by Amvex. If this device is subject to improper maintenance, repair, use and or abuse leading to malfunction of the device, replacement is the sole responsibility of the user.

Intended Use

Once the product is received, open the package and inspect the item for any damage. If damage is evident, contact your representative immediately and do not use the product.

******It is very important to allow product to remain in original packaging for 24 hours to acclimatize to room temperature before use******

After thorough inspection of the product, please note any and all caution and warning labels in this manual. The Hose Assembly is designed for gas distribution in centralized areas of hospitals, dental clinics and veterinary facilities. Our Medical Hose Assembly is available in many different styles and combinations, made according to your specifications. They are to be used under the direction of a health care professional.

NOTE: When storing the Hose Assembly, ensure it is kept in a sealed package to avoid environmental damage. Store in a clean and dry environment to prevent damage.

Specifications

The Hose Assembly is CE Marked and manufactured with medical grade PVC tubing. The tubing is reinforced with nylon braiding. All ingredients in the formulation of the tube comply with FDA, NFPA 99 and CSA Z305.2 requirements. The maximum working pressure is 200PSI @ 70° F. The fittings are permanently crimped on each end with ferrules.

⚠ WARNING: Prior to administering medical gases, confirm required gas/suction.

TO MINIMIZE THE RISK OF EXPLOSION OR FIRE:

- Never attempt to attach a Hose Assembly directly to a cylinder.
- Never use grease, oil, organic lubricants or flammable materials on or near the Hose Assembly.
- Never smoke in an area where Oxygen is being used.
- Never use any type of flame or flammable or explosive material near the Hose Assembly.
- Always follow CGA, ANSI and EN739 standards for Hose Assemblies and Medical Gas Products (E-7) and Oxygen Handling (G-4).

Operating Instructions

1. Inspect the Hose Assembly for damage. If any is found, do not use the Hose Assembly.
2. Connect the Hose Assembly to the correct mating gas specific outlet. The Hose Assembly is gas specific according to the fittings permanently crimped on.

CAUTION: Only people that have been properly trained on the use of the Hose Assembly, and have been instructed to, should use this product. Keep the Hose Assembly in a clean area when not being used. Ensure that all connections are tightened and free of leaks prior to use. Use only an Oxygen-safe leak detector when testing for leaks. DO NOT gas sterilize with ETO. DO NOT clean with pungent hydrocarbons.

NOTE: The flow may be affected by the attachment of multiple hoses.

CAUTION: Connection to the gas source must be done by using only the appropriately indexed fitting.

Cleaning Instructions

⚠ WARNING: DO NOT submerge product in any form of liquid. This will cause damage and void the warranty. Wipe outside of product with a clean cloth, which is dampened with a mild cleaning solution.

Troubleshooting

Contact your dealer or the technical support department at Amvex Corporation for assistance if the Hose Assembly does not function.

Maintenance Prevention

Inspect the product before and after use for any damage and for ease of operation.

MRI WARNING: This product contains magnetic, ferrous material that may affect the result of an MRI, unless the product is ordered as MRI conditional.

WARRANTY

Amvex Corporation warrants its Medical Secondary Equipment to be free from defects in material and workmanship for a period of five (5) years from the date of shipment. Within the first twelve (12) months from date of shipment Amvex will repair or replace any part which is proven to be defective at Amvex's cost. After the first twelve (12) months, Amvex will send the parts to the customer free of charge, but the shipping and installation will be borne by the Customer. This warranty is valid only when the product has been properly installed according to Amvex specifications, used in a normal manner and serviced according to factory recommendations. It does not cover failures due to damage which occurs in shipments or failures which resulted from accidents, misuse, abuse, neglect, mishandling, alteration, misapplication or damage that may be attributable to acts of God.

AMVEX CORPORATION DOES NOT HONOR VERBAL STATEMENTS CONCERNING THE WARRANTY.

The distributor and/or dealer is not sanctioned to create verbal warranties about the product described in this agreement. Any statements will not be honored or be made part of the agreement of sale. This document is the final, complete and exclusive terms of the agreement.

THIS WARRANTY IS EXCLUSIVE AND REPLACES ALL OTHER WARRANTIES.

Amvex Corporation shall not, under any circumstances, be liable for incidental or consequential damages including, but not limited to, profit loss, loss of sales or injuries to person(s) or property. Correction of non-compliances as noted above will result in completion of all liabilities of Amvex Corporation whether based on agreement, neglect or otherwise. Amvex Corporation reserves the right to stop manufacturing any product or change materials, designs or specifications without notice.

All claims for warranty must first be approved by Amvex's Service Department: (support@amvex.com or 905-764-7736 Toll Free: 1-866-462-6839). A valid Return Goods Authorization (RGA) number must be obtained from Amvex prior to the commencement of any warranty claim.

Authorized Representative
in the European Union:

EC REP

Oxygen Care Ltd.
2 Holfeld Business Park
Kilmacanogue Co Wicklow
Ireland